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INSTRUCTION MANUAL ELECTRIC PATIENT LIFT FORTUNA

INDEX

1. Warnings

2. Parts of the Lift

3. Specifications

4. Assembly and Operation

5. User guide

6. Battery and Battery Charge

7. Care and Maintenance

8. Troubleshooting

9. Indications and contraindications

Please read this owner's manual carefully before operating your electric patient lift and keep it for future reference

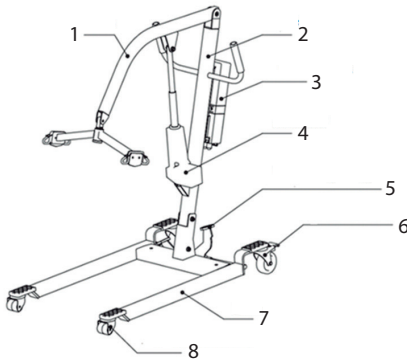
1. WARNINGS

- 1.1. Assemble the product according to user's manual.
- 1.2. Properly fix the actuator onto electric patient lift.
- 1.3. Charge the battery no less than 4 hours until fully charged.
- 1.4. Read user's manual carefully until it is fully understood.
- 1.5. For the sake of safety and user's need, please select the right model, size, material, slings and accessories.

Notice

Notice to the user and/or patient: any serious incident which has occurred in connection with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.




2. PARTS

1. Lifting arm
2. Frame
3. Control box
4. Actuator
5. Width adjusting pedal
6. 4" Rear castor with break
7. Base
8. 3" front castors

3. SPECIFICATIONS

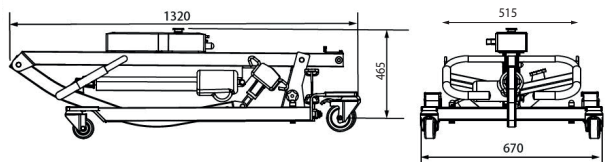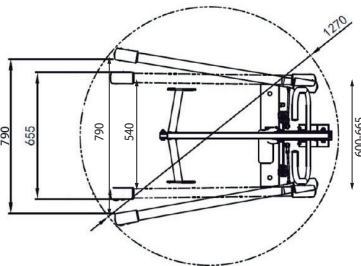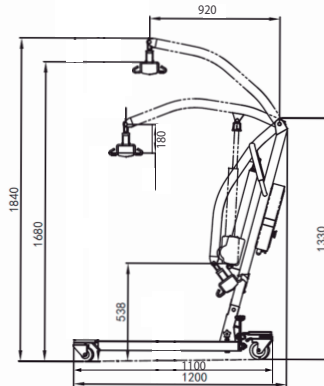- a. Product Size: 1200x670x1330mm (LxWxH)
- b. Maximum Fork Range: 765mm
- c. Lifting Range: 550mm (lowest), 1680mm (highest)
- d. Max speed at no load: 31mm/s
- e. Max speed at full load: 6.7mm/s
- f. Input power: DC24V
- g. Motor: 24V/8000N
- h. Load Capacity: 135kg
- i. Net weight: 45kg
- j. Noise: 65dB(A)
- k. Normally these battery can sustain lifting up and down not less than 60 times after full charging, also this model cannot work while it is charging

4. ASSEMBLY AND OPERATION


4.1. Inspection

Please verify if there are missing parts (checklist for accessories shall be subject to list attached with the product).

Description	Quantity	Image
Frame	1 set	
Control box	1 pc	
Actuator	1 pc	
Controller + cable	1 set	
Open spanner #16 and #17	1 pc	NA

Step 3 : Installation of control box


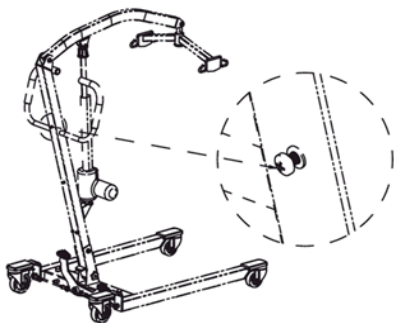
1. Fasten the 2xM5x10 screws into threaded holes on the frame as in the following picture:

Step 4: Remove and install the battery box.

1. Removal: press down the switch A and remove the battery following the direction of B (Fig. 1).

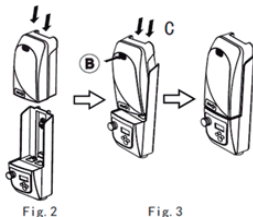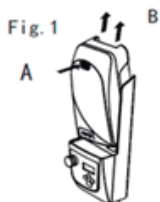2. Installation: Put the battery in place and press the switch along the C direction. Make sure that the battery does not separate without pressing the switch (Fig. 2 and 3).



4.2. Installation procedure

Step 1: Release the locknut and bolt and the raise up the frame and align the holes. Fix bolt as shown in drawing on the right and have not properly attached (make sure no movement on the frame).


Step 2: Release the strap.

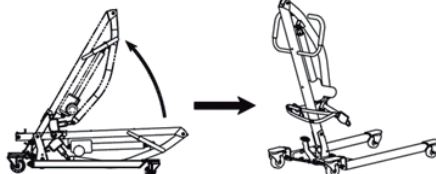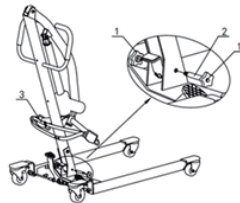1. Locknut
2. Bolt
3. Strap

2. Rotate the lock to direction of open. Fasten the control box into the frame and have the upper screws locked.

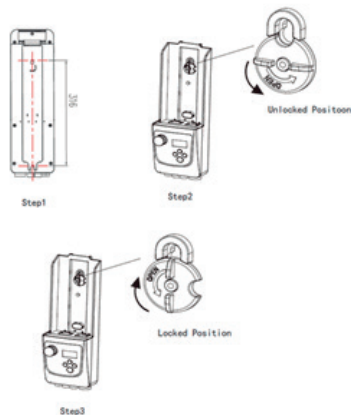
3. Turn the lock 90 degrees to the right and lock the control box.

5. USER MANUAL

5.1. Use of controller

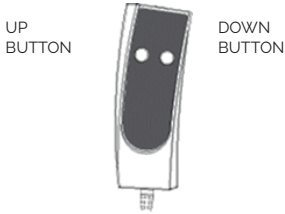Control the upper and lower part of the lift arm by means of the hand control's up and down buttons. The patient lift stops working once the up and down buttons are released.

5.2. Base width adjustment

The width of the base can be adjusted with the foot pedal. Press the pedal to the left for a wider base and press the pedal to the right for a narrow base.

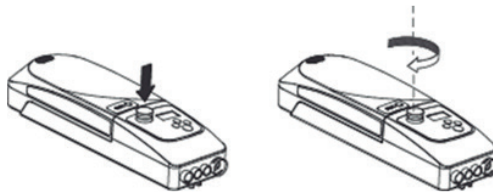
5.3. Emergency stop button

The emergency stop button is used to switch the power on/off. It allows the engine to stop running immediately in case of danger.

Off: press the red button (see picture A).


On: turn the red button clockwise (see picture B) and the control device will be activated.

It is recommended that this button be pressed if the lift has not been used for more than 3 days, to prolong the battery life.


5.4. Emergency ascent and descent

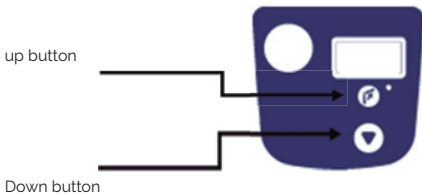In case of malfunction at the controller or no controller, press the up/down button to return the actuator to its place.


5.5. Control device connection

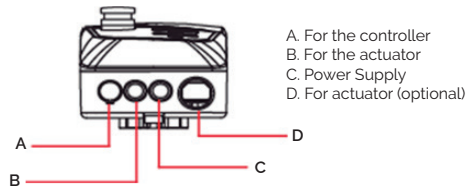
There are 4 ports on the control device, 1 for the controller, 2 for the actuator and the rest for the power supply.

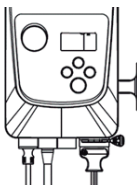
5.5.1. Insert 4 cables into 4 different ports as appropriate one by one.

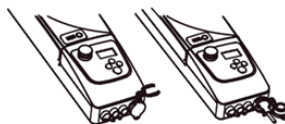5.5.2. There is a plastic cover which is used to cover the power port after charging.

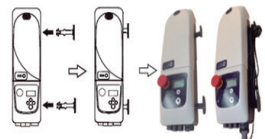
5.5.3. After charging, fix the cable on the "T" knob.

5.6. Manual emergency descent and manual emergency ascent

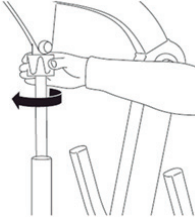Turn the red sleeve along the direction of the arrow for manual emergency descent and the reverse for manual ascent.


Caution: It is not necessary to lock the rear wheels during a raising and lowering movement.

Warning: locking the wheels during raising the patient increases the risk of tipping during the raising and lowering movement.


5.8. Patient lifting operation

5.8.1. Security operation

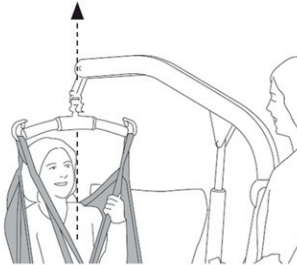Keep the lift arm and body in an upright position. Warning: an unbalanced lift creates the risk of tipping over.

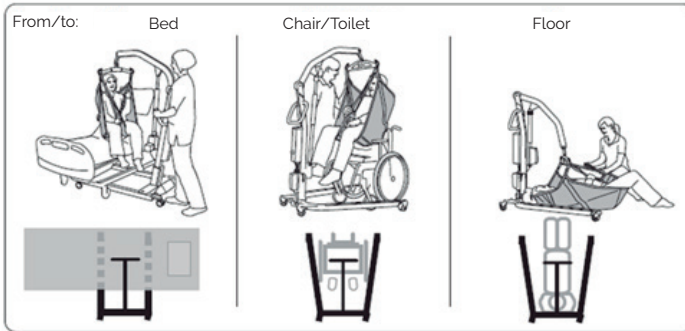
5.8.2. Lifting the patient

Note: A pillow under the patient's head will make it more comfortable.

5.8.3. Folding structure

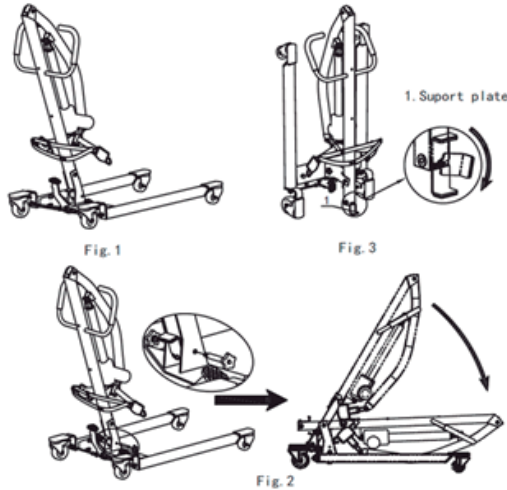1. Lower the actuator and then fully secure the lift arm.
2. Remove the bolt before folding the frame.
3. The support plate must be released before the frame is lifted.

Note: Remove the control box before folding the structure.

6. BATTERY CHARGE

1. When the battery has only 1 bar left, the controller box will beep 4 times, reminding you to charge immediately. The actuator will not operate without charging.
2. Battery charging procedure.
When the battery beeps, turn off the emergency stop button and plug in the controller box. Once the battery is fully charged, the battery level status shows 4 bars (100%) and the LED on the controller box will stop flashing.

Notes

1. Turn off the emergency stop button and stop using the lift during loading.
2. Remove the control box and keep it in the shade, if the patient does not use it for a long period of time. Charge the battery once a month.
3. Do not charge the battery in a humid environment.
4. In case it is not used daily, it is recommended that you turn off the emergency stop button to maintain the battery level.

7. CARE AND MAINTENANCE

For safety reasons, the inspection routine listed below should be carried out daily.

1. Check for any damage or wear
2. Make sure that the nuts and bolts on the lift arm are secure.
3. Check if the hook on the lift arm is working properly.
4. Check the controller, actuator and base width adjustment.
5. Check emergency lowering function.
6. Verification of the emergency ascent function.

If necessary, use warm water or alcohol to clean the lift. Make sure that there is no dust and hair on the wheels.

Caution: When cleaning the lift, do not use products containing phenol or chlorine, as this may cause damage to the plastic material.

Transport and storage

Press the emergency stop button for transport or if the lift will not be used for a certain period of time. The temperature for storage of the lift should be above 0° and relative humidity less than 60%.

Inspection

Make a complete inspection at least once a year. Pay close attention to worn parts, connections and screws.

8. TROUBLESHOOTING

DESCRIPTION	SOLUTION TO THE PROBLEM
If the lift doesn't work properly	1. Check if the emergency stop button is pressed. 2. Verify that the controller and actuator wires are properly connected. 3. Check the battery level on the display screen on the control panel. 4. Check for any problems or damage to the battery contact panel.
If the charger doesn't charge the battery	1. Check if the charger is plugged in correctly. 2. Check if there is any problem or damage to the battery contact panel.
If the lift stops during the ascent	1. Check whether the emergency stop button is pressed in. 2. Use the emergency lowering.

9. ERROR CODE / TROUBLESHOOTING

DISPLAY	INFORMATION	DELETION
	Low battery (less than 20 V DC, lift actuator can only retract).	Charge the battery pack at once.
	Lift actuator overloaded (after 4 minutes of continuous lifting).	Stop operation for a while. The error message will disappear automatically after a sufficient rest.
	Lift actuator overloaded.	Find out and remove the blockage or inadequate load.
	Spread the actuator over the load.	Please find out the cause (let the actuator move in reverse direction for 2 seconds).
	Cumulative operating time of more than 600K seconds on the lifting actuator.	Call agency to replace actuator.

10. INDICATIONS AND CONTRAINDICATIONS

Indications

All patients whose mobility is limited but not mentioned in the contraindications.

Contraindications

Disorders such as Osteogenesis imperfecta, osteoporosis or spinal column damage and mental aberration or epileptic attacks can be contraindications. HY101-01A (HY201) only permitted to be used after careful consideration of the individual patient by the doctor and the care personnel. Meanwhile, the use of a patient lift should be avoided if the patient is agitated, resistant or combative.

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GUANGDONG KAIYANG MEDICAL TECHNOLOGY GROUP CO., LTD.
SUNGO EUROPE B.V.
GRUPO R. QUERALTÓ S.A.